DE GEBRAUCHSINFORMATION

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EN PACKAGE LEAFLET

Please read this information sheet carefully as it contains important content and details about the tampons.

Tampon strength

Normal Absorbency: 9-12 g, with normal, moderately heavy menstruation.

Careful menstrual hygiene is fundamentally important

Good menstrual hygiene is also fundamentally important and advisable to prevent infections with other germs. The following information is given in textbooks (2) or by manufacturers of hygiene products:

Tampon use

- Before and after inserting a tampon, you should wash your hands thoroughly and remove carefully the protective cover from the tampon.
- When menstruation begins, it is advisable to start with the smallest tampon thickness.
- The strength of the tampon should be adjusted to the current flow strength.
- On days with heavier bleeding, the tampon may need to be changed more frequently, but usually every 3-6 hours.
- On days with light bleeding, a smaller tampon can be worn for 6-8 hours.
- Manufacturers recommend changing the tampon before it has been in use for eight hours.
- The same tampon size can be used at night as during the day. Manufacturers recommend inserting the tampon be-

- fore going to bed and changing it as soon as you get up.
- If you sleep for more than eight hours, manufacturers advise you to change the tampon as soon as you get up.
- It is important to only use tampons from undamaged original packaging.
- Never use more than one tampon at a time.

Source: www.frauenaeerzte-im-netz.de; (2) Paediatric and adolescent gynaecology, Patricia G. Oppelt/Helmuth-Günther Dörr (eds.) 2014

Disposal

The tampon is wrapped well and disposed of with normal household waste or the waste bin in the sanitary area.

DANGER

- If the cord is no longer accessible from the outside and the tampon cannot be removed as a result, a doctor must be consulted.
- If you experience an allergic reaction or irritation from using a tampon, stop using the tampon and consult a doctor.

About Toxic Shock Syndrome (TSS)

In rare cases, symptoms such as high fever over 102.2°F, vomiting, diarrhoea, dizziness or skin rash can occur during menstruation. Although some harmless illnesses start the same way, one specific bacterial infection – toxic shock syndrome (TSS) – can have life-threatening consequences. If the symptoms described occur, the tampon must be removed and a doctor consulted. The doctor must be informed about the current menstruation. We advise women who have experienced TSS to consult a doctor before continuing to use a tampon.

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